

QUICK SET-UP GUIDE

STEP 1

Plug the AC power adapter into an electrical outlet that is **not controlled by a light switch**.




STEP 2

Insert the T adapter into a telephone jack that is centrally located. The phone cord from your home phone can be connected to the empty side of the T adapter.



STEP 3

Turn the ON/OFF switch on the back of the console to ON (always leave it in the OFF position when the power is not plugged in to avoid draining the battery).

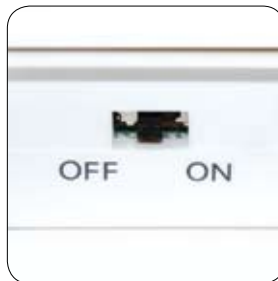

STEP 4

Test the system by pressing the black button on the transmitter. (Please refer to our "How to test and use your Lifewatch system" on other side of this page.)



Key controls and indicators are identified here.

Your console and transmitter are already programmed and ready to use. A battery inside the console provides up to five days of back-up power.

Speaker: Transmits two-way voice connection with certified EMT's at the Lifewatch Center.

LED indicator: Lights green when power is on.

Green CANCEL/RESET button: Push to cancel a false alarm.

Microphone: Powerful enough to transmit your voice from across the room to the Lifewatch Response Center.


Red ALARM button: For triggering an emergency call to the Lifewatch Response Center.



T-Adapter: The phone cord from your home phone can be connected to the empty side of the T adapter.

LED indicator: Lights red when alarm is successfully transmitted to PERS console.

Black ALARM button: For triggering an emergency call to the Lifewatch Response Center.

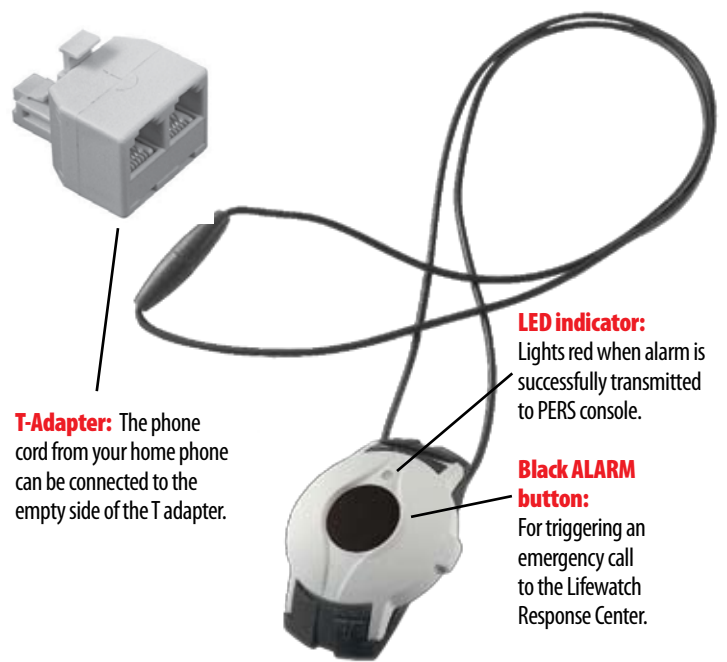

DSL - requires a filter between telephone jack and our T adapter.
VOIP - May require special equipment. Please contact us if needed.